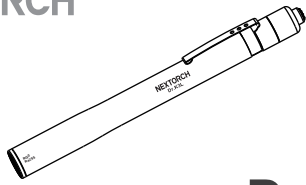
NEXTORCH

ENGLISH

Learn more on  
www.nextorch.com

Dr.K3L

SPECIFICATIONS

Bulb	Dual-light sources (Yellow + White)		
Ra	≥95		
Photobiological safety	RG0 (Yellow)		
CCT	3000K (Yellow)	5500K (White)	
Modes	Low	Medium	High
Light Output	2 lm	10 lm	75 lm
Run Time	42 h	15 h	1 h 30 min
Beam Distance	7 m	15 m	40 m
Battery	2 × AAA Batteries		
Impact Resistant	1 m		
Weight and Dimensions	Weight: 27.8 g (Without battery); Size: 149.8 mm (Length) × 14.3 mm (Head Diameter) × 13.3 mm (Tail Diameter)		
Material	Aluminum Alloy 6061-T6, Type II Anodized		
Accessories	2 × AAA Batteries		

Above data were tested with 2 × AAA batteries at 22°C ± 3°C. The specifications might be different when using different batteries or testing in a different environment.

FEATURES

- Yellow and white dual-light sources, low- medium-high three light modes to meet various needs.
- Blue light hazard class, RG0 (Risk Exempt).
- High color rendering, Ra ≥ 95, uniform light spot design for better observation.

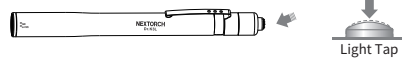
INSTRUCTIONS

Battery Installation


ON / OFF

Momentary on / off: half press for momentary on and loosen to turn off.

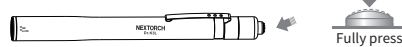
Constant on / off: click for constant on and repeat to turn off.

Mode Switch

Three modes: Low (Yellow) - Medium (White) - High (White)

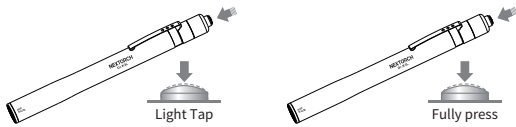1. In the OFF state, half press to preview modes, continuously tap to switch modes and fully press to lock the mode;
2. Click the button to switch modes in sequence.

Smart Memory Function

Turn off the penlight for more than 2s, turn it on again, it will enter the last working brightness mode it has memorized.

MAINTENANCE

1. Being infected with seawater or any corrosive chemicals, please rinse immediately with clean water.
2. Please use high quality batteries; Remove batteries when not working for a long time, and then store in a cool dry place.
3. If the waterproof O-ring is damaged, replace it immediately.

WARRANTY

1. NEXTORCH offers 5-year warranty.
2. NEXTORCH warrants our products to be free from any defects in workmanship and material, we will replace or return faulty item. NEXTORCH reserves the right to replace similar products if original one is discontinued.
3. The warranty excludes other accessories, but rechargeable batteries are warranted for 1 year from date of purchase.
4. Any accessories or products are not covered in the warranty, NEXTORCH can repair for users with reasonable fee.
5. Please scan the below QR code and access to NEXTORCH website (www.nextorch.com) to register your purchasing.  
Email us info@nextorch.com  
Call us: 0086-662-6602777  
Or contact local dealer

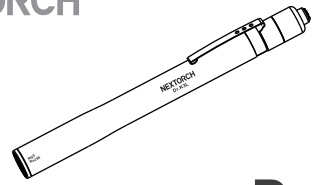
CONTACT WITH NEXTORCH DESIGNER

In order to improve NEXTORCH, we appreciate you could offer our designers your after-use feedback and creative suggestions by scanning the following QR code. Thank you!

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